### Praxis

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En

**Praxis**

Praxis, system for finishing and polishing composite restorations is composed by abrasive discs and a pressure mandrel, which provides agility at side inversions, as well as in disc replacements. Available in 3/8" and 1/2" diameters and four grits, identified by colors.

**Use:** Install the mandrel at the counter angle. Fit the disc onto the mandrel. For better results, use the four grits, successively from the coarsest to the finest one. Product must be applied dry, with low to medium speed and with intermittent touches. It is recommended to use 10.000 rpm for the coarsest discs and 30.000 rpm for the finest.

**Physical principle of operation:** The abrasive surface of the disc with friction on the tooth provides the finishing and polishing the restored area.

**Warnings, precautions and restrictions:** Professional use only. Discs must be disposed after each patient. Its reuse may cause cross contamination. Prolonged contact of the rotary instrument with the dental tissue may cause heating and jeopardize pulp vitality. In order to avoid accidental swallowing of the product it is necessary the total isolation of the operatory field. The user will be entirely responsible for the combined use of the mandrel or the disc with similar products. If there is proven allergy to any of its components, product must not be used.

**Connexion with counter angle:** Praxis mandrel is manufactured in compliance with specifications from the rule ISO 1797-1.

**Sterilization:** The mandrel may be sterilized by autoclave at 126°C, 147kPa, 16min. Before sterilization, proceed with the cleaning using water and enzymatic detergent.

Ref.	Code	Content / Contenu
3001	0002000034	120 assorted discs + mandrel / 120 disques assortis + mandrin/ 120 Scheibchen sortiert + dorn
3002	0002000035	60 assorted discs + mandrel / 60 disques assortis + mandrin/ 60 Scheibchen sortiert + dorn
3003	-	Refil 3/8" 30 discs (assorted/G/M/F/XF) / Refil 3/8" 30 disques (assortis/G/M/F/XF) / Refil 3/8" 30 Scheibchen (sortiert/G/M/F/XF)
3004	-	Refil 1/2" 30 discs (assorted/G/M/F/XF) / Refil 1/2" 30 disques (assortis/G/M/F/XF) / Refil 1/2" 30 Scheibchen (sortiert/G/M/F/XF)
3007	-	Refil 3/8" 50 discs (assorted/G/M/F/XF) / Refil 3/8" 50 disques (assortis/G/M/F/XF) / Refil 3/8" 50 Scheibchen (sortiert/G/M/F/XF)
3008	-	Refil 1/2" 50 discs (assorted/G/M/F/XF) / Refil 1/2" 50 disques (assortis/G/M/F/XF) / Refil 1/2" 50 Scheibchen (sortiert/G/M/F/XF)
3009	-	Refil 3/8" 85 discs (assorted/G/M/F/XF) / Refil 3/8" 85 disques (assortis/G/M/F/XF) / Refil 3/8" 85 Scheibchen (sortiert/G/M/F/XF)
3010	-	Refil 1/2" 85 discs (assorted/G/M/F/XF) / Refil 1/2" 85 disques (assortis/G/M/F/XF) / Refil 1/2" 85 Scheibchen (sortiert/G/M/F/XF)

**Composition:** Aluminum oxide , polyethylene terephthalate, synthetic rubber resin, aluminum, Water-based pigments. Mandrel: Metal.

**Storage:** To be stored in a dry place, sheltered from sunlight.

Version 01 - 03/2017

Burgundy	Coarse
Dark pink	Medium
Light pink	Fine
White	X-Fine

**TDV Dental Ltda**  
 CNPJ 01.591.786/0001-60  
 XV de Novembro, 9944 | CEP 89107-000  
 Pomerode/SC - Brasil  
 Reg. ANVISA 10291220030  
 Resp. Técnic. Tais Cilogla C. Pereira | CRF/SC 13884  
 Uso profissional / Uso profissional / Professionnel / Gebrauch  
 Professional use only / Usage professionnel / Uso professionale.  
 ☎ 55 47 3395-6115 | 0800 47 1020  
 info@tdv.com.br | www.tdv.com.br

**EC REP**  
 Cobis SA  
 Bc. General Wahn 53  
 13030 professional, Biotum  
 Tel. + (33) 2732-5054  
 E-mail: mal@cobis.net

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