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c.0 m.0 y.0 k.80

Pantone 425C

Material

Impressão: 1x1

Papel: Sulfite 63g/m²

Dobra: 3 Vincos Horizontais | 1 Vinco Vertical

Tamanho Aberto: 78 x 116 mm

Marcas de Corte/Dobra

——— Corte Linear

- - - - - Dobra

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W 17 - 06/2022

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Anatomic Wedges / Interdental Wedges

TDV wooden Wedges are indicated for general use in restorative procedures, with the mechanical function of adapting matrices and/or separating adjacent teeth. The wooden wedges do not require cuts or adjustments, because its shape allows an easy insertion into the interproximal areas and ensures a perfect adaptation of the matrices.

Use: Select the most appropriate wedge size for the procedure. Insert the wedge into the interproximal space. Once the procedure is completed, the wedge must be removed.

Warnings, precautions and restrictions: Professional use only. Single use only. Its reuse may cause cross contamination. If there is proven allergy to any of its components, the product must not be used. To avoid an accidental swallowing of this product, it is necessary the application of a total isolation technique of the operator field. The product has a sharp point and must be handled with caution to avoid risk of causing harm to the patient. Any serious incident related to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

Composition: Natural: Wood. Colored: Wood and non-toxic reactive dyes.

Storage: To be stored in a cool and dry place, protected from sunlight.

Disposal: Dispose in accordance with applicable local requirements.

Ref.	Code	Content
0.4952C	00004020022	Anatomic wedges - 500 assorted units (colored by sizes)
0.4953	0004020023	Anatomic wedges n°1 yellow - 100 units
0.4957C	0004020024	Anatomic wedges - 100 assorted units (colored by sizes)

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EC REP: Obelis S.A.