## Cold Liner Reembase

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EN

### Cold Liner Rebase

Cold Liner Rebase is a rigid self-curing resin (powder and liquid), developed for permanent relining of full and partial, removable dentures, plaques for bruxism and orthodontic appliances. It can be applied directly in the patient's mouth, because its polymerization occurs at a temperature of approximately 37°C, and does not attack the oral mucosa. Cold Liner Refill meets the requirements of ISO 20795-1, and its viscosity allows fluidity and quick drying, ensuring high quality results. The kit comes with Glaze, Insulator and meters. The Glaze is indicated for waterproofing of the resin, promoting brightness and protection against pigmentation and accumulation of bacteria, since it seals the porous surface of the pieces. The surface Insulation is indicated for insulating the external part of the prosthesis or orthodontic appliance, preventing the product from sticking in unwanted areas.

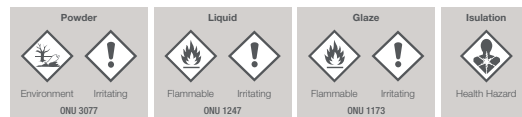
**Composition: Powder colorless:** Polymethacrylate. **Powder pink:** Polymethacrylate and pigment. **Liquid:** Methacrylates, plasticizer, initiator and essence. **Glaze:** Polymethacrylate, ethyl acetate and essence. **Insulation:** Liquid vaseline.

**Indication:** Indicated for the relining of total and partial dentures, removable, plaques and orthodontic appliances. It can be applied directly into the patient's mouth.

**Storage: Powder:** Store in a dry, cool place, well ventilated, sheltered from sunlight. **Liquid / Kit / Glaze / Insulation:** Store in a dry, cool place, well ventilated, sheltered from sunlight. Do not store the product in temperature above 30°C (86°F).

**Use: Preparation:** Proceed with an induced relief of the denture to be rebased, removing approximately 1.5mm of resin, with the help flow rotation finishing device drills. Clean and dry the piece. In the case of dentures, total or partial, cover the surfaces of the teeth, gingival and palatine with insulation. Do not apply coating within 3mm (1/8 inch) of the peripheral border. **Resin Mixing:** Shake the powder vial before use. First place the liquid in the Dappen cup, then add the powder, gradually. The proportion of Cold Liner Rebase TDV resin components is 3 parts powder to 1 part liquid (v/v). Mix it for 30 seconds tops, avoiding the formation of bubbles (it is recommended to use the Spatula n° 36). Wait for the plastic stage to be applied to the part. **Application of full and partial dentures:** Spread Cold Liner Rebase TDV resin over the area to be covered. Insert the prosthesis onto patient's mouth and ask him to close his mouth gently until occlusion. Wait 3 minutes and ask the patient to move the lips and maxillae in order to obtain a muscular adaptation in the peripheral area. Remove the prosthesis from patient's mouth and rinse it with cold water. Remove excess material. Insert the prosthesis onto patient's mouth and ask him to firmly close the mouth in occlusion, hold this position steady for 5 minutes. Remove the prosthesis from patient's mouth and rinse it with cold water. **Application in plaques and orthodontic braces:** Spread Cold Liner Rebase TDV resin over the area to be covered. Insert the piece onto patient's mouth, keeping the piece in steady position for 4 minutes. Remove it from the patient's mouth and rinse it with cold water. Remove material residues. **Finishing and Polishing:** After completion of the polymerization (10 to 15 minutes), proceed the finishing in the conventional way. It is recommended to use a tungsten drill bit and then a sandpaper disc to smooth the surface. Clean the piece with water and detergent and dry the entire surface using oil-free compressed air. Carry out the polishing in the conventional way with the assistance of a polishing paste or abrasive rubbers. Apply two to three layers of glaze on the piece. Wait for 2 to 3 minutes. Wash the piece with water and detergent. In case of complete or partial prosthesis, do not pass the glaze over other parts of the denture other than the rebasing material. **Glaze:** After finishing and polishing the piece, apply two to three layers of glaze over the rebased area. Wait 2-3 minutes and wash the piece with water and detergent. In the case of complete or partial dentures, do not pass the glaze over the thermopolymerizable resin.

### Warnings, precautions and restrictions:

### DANGER / WARNING

For dental use only. Professional use. Handle the product in a ventilated place, using a spatula, protective clothing, disposable gloves, goggles and a mask. The product is recommended to be handled at room temperature (20°C to 25°C). Follow the usual procedures for microbiological control during handling. Containers, instruments and hands must be clean to avoid contamination of the parts and the patient with micro-organisms. Containers, instruments, parts and hands must be free of wax or grease in order to provide an adequate chemical bond between the resins. After relining, a treatment in water at 55°C during 10 minutes is recommended. Follow the instructions for use to avoid monomer residue on the manufactured part, color change, wear and tear or breakage. The failure in complying with the instructions for use may affect product performance. If the patient is known to be allergic to any of the components, the material must not be used. Handling of this product is not recommended for pregnant or breastfeeding women. The temperature reached by the resin during the polymerization is approximately 37°C (98.6°F), and may exceed in layers of very thick resin (greater than 2mm). Do not reuse containers. Store in a locked place. **Resin:** It may produce flammable air / powder mixtures. **Disposal:** Waste materia, content and container must be disposed in an industrial or sanitary

landfill according to local legislation enforcement. Do not dispose the captioned waste in sewers, waterways or regular household waste. The Chemical Products Safety Data Sheet for such product may be requested by phone. Flammable product. Ignition temperature of the liquid: 421.4°C. Flash point: 10°C. Keep away from any sources of ignition - Do not smoke. **Liquid / Powder / Glaze / Kit:** Wash hands thoroughly after handling. Store in a well-ventilated place. Keep in a cool place. If you feel unwell, please contact a TOXICOLOGICAL INFORMATION CENTER or a doctor. **Liquid / Powder / Kit:** Causes skin irritation. May cause allergic skin reactions. May cause irritation of the respiratory tract. Avoid inhaling mists, vapors and dust. Use only outdoors or in a well-ventilated area. Contaminated clothing must not leave from its workplace. Avoid release to the environment. In case of contact with skin: Wash with plenty of soap and water. In case of inhalation: Remove the person to a ventilated area and keep at rest in a comfortable breathing position. If skin irritation or rash occurs: Get medical advice/attention. Remove all contaminated clothing and wash before reuse. **Liquid / Glaze / Kit:** Highly flammable liquids and vapors. Keep away from heat, sparks, naked flames and hot surfaces. - Do not smoke. Keep container tightly closed. Proceed with grounding on containers and product receiver during shipment. Use explosion-proof electrical, ventilation, and lighting equipment. Only use anti-sparking tools. Avoid the accumulation of electrostatic charges. In case of contact with skin (or hair): Immediately remove all contaminated clothing. Rinse skin with water or take a shower. In case of fire: For extinguishing it use chemical powder, carbon dioxide (CO2), Halon, water mist, or alcohol foam. **Liquid / Kit:** Toxic to aquatic organisms. Collect the spilled material. **Powder / Kit:** Toxic to aquatic organisms with long lasting effects. Avoid inhaling dust. **Glaze / Kit:** May be harmful if swallowed. Causes serious eye irritation. In case of contact with eyes: Rinse thoroughly with water for several minutes. In case of wearing contact lenses, remove contact lenses, if easy to do. Continue rinsing. If eye irritation persists: Search for medical advice. **Insulation / Kit:** May be harmful if swallowed and enter the respiratory tract. If swallowed call a poison center or doctor. Do not induce vomiting. Any serious incident related to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

**Support material:** Measuring devices.

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