Pt
Cunhas Elásticas

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En

Elastic Wedges

TDV Elastic Wedges are indicated for general use in restorative procedures, compatible with all matrix systems. With the mechanical function of adapting matrices and/or separating adjacent teeth, they act simultaneously on both, lingual and vestibular sides. The wedges are available in three different sizes coded by colors, to be chosen according to the cervical-occlusal height of the proximal box.

3,2 mm

Blue - indicated for conservative cavities.

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2,6 mm

Yellow - indicated for medium cavities.

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Use: After installing the matrix, select the most appropriate wedge size for the procedure. 1 - Insert the wedge in the Unimatrix Plier through its 2 larger holes. 2 - Stretch the wedge until the central part reaches the thinnest thickness to slide between the matrix and the adjacent tooth. 3 - Position the wedge into the interdental gap and remove the pliers. 4 - After completing the procedure, pull the wedge by one of its extremities and cut it out for removal.

Composition: Rubber and atoxic colorants.

Storage: To be stored in dry and fresh place, protected from sunlight.

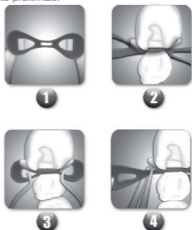
Warnings, precautions and restrictions:

Professional use only. Single use only. Its reuse may cause cross contamination. If there is proven allergy to any of its components, the product must not be used. To avoid an accidental swallowing of this product, it is necessary the application of a total isolation technique of the operator field. The product is a medical device. Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

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Resp. Técnic.: Gisele Karina Gruber - CREA/SC 179557-7

Reg. ANVISA 10291220066

Professional use only / Usage professionnel.

Check instructions for use /

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☎ 55 47 3395-6115

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Medical Device
Dispositif Médical