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c.0 m.0 y.0 k.80
Pantone 425 C

Material

Impressão: 1x1

Papel: Sulfite 56g/m²

Dobra: 1 Vinco Vertical

Tamanho Aberto: 135 x 190 mm

Marcas de Corte/Dobra

———— Corte Linear

- - - - - Dobra

PT

Contact + Gold

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EN

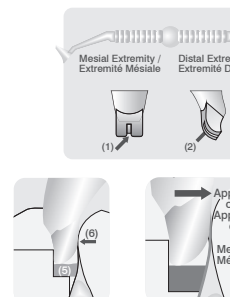
Contact + Gold

Instrumentos for contact point reconstruction

Contact + Gold is a unique instrument developed to easily reconstruct consistent and precise contact points with ideal shape and position in class I restorations with composites. The instrument has its extremities designed to obtain the desired effect with simple application of force against the adjacent tooth. The slot (1) in the convex surface (2) produces a resin wall that once polymerized forms the contact point. Contact + Gold is presented in a kit with two instruments, one for small cavities (3) and the other for large cavities (4). Each instrument has a mesial (3) and a distal (4) tip.

Use: Perform preparation by inserting the matrix/wedge set, giving preference to sectional matrices. Apply enough resin into the cavity to envelop the tip of the instrument (5). Select the appropriate instrument for the procedure, considering the size and position of the surface to be restored. Insert the instrument into the cavity to control the position of the convex surface in relation to the desired location of the contact point (6). Move it against the matrix and the adjacent tooth, according to the face to be restored: mesial (7) or distal (8) and as indicated by the arrow printed on the end (Push = Push). Cure the resin holding the instrument in place, applying moderate force. Remove the instrument from the cavity. The resin that was in the fissure, now cured, can preserve the contact. Complete the cavity filling process, using the usual procedures.

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Ref.	Code	Presentation / Presentation
0.42026	0005020002	Contact + Gold - 2 instruments for contact point reconstruction Contact + Gold - 2 instruments pour obtention du point de contact

Cleaning: Carry out the cleaning with the aid of water and enzymatic detergent 4 enzymes (Amilase, Protease, Lipase and Carboxidase) and with pure pH range between 6.5 - 7.5. Prepare the dilution solution according to the manufacturer's instructions. The water should be potable. For best effectiveness, it is recommended that the temperature be between 40°C and 55°C. Submerge the products and leave them in contact with the solution for 5 minutes. Remove parts and rinse under running water for at least 10 seconds. Dry the piece with absorbent paper or disposable wipes. Sterilization: The instrument should be autoclaved (126°C, 16 min) after each patient. Sterilization performed with any method other than autoclave can damage the product. Repositioning Limits: This is a reusable device. The manufacturer guarantees up to 200 reuses. Inspection: Check the product before each use, observing whether there is damage to the tips or cracks along the instrument, and do not use in such cases. Cracks are indicative of material fragility, which can cause the product to break.

Warnings, precautions and restrictions: Professional use. If there is proven allergy to any of its components, the product should not be used before approval by the manufacturer in autoclave before use. Any serious incident related to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

Storage: To be stored in dry and fresh place, protected from sunlight.

Disposal: Dispose in accordance with applicable local requirements.

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REC IREP Obelis S.A.