



Prophylactic Strip is an indispensable aid in prophylaxis procedures, since it easily accesses the interproximal region, where its active face effectively provides the removal of dental calculus. The circular micro-blades, located around the holes in one face of the strip, has as main characteristic the action against calculus, thus preserving the enamel and restorative materials integrity.

Indications: Prophylactic Strips is indicated for clinical use, removing dental plaque and tartar.

Use: Insert the strip in the interproximal space, placing its active face turned to the desired tooth. Hold the strip by the ends, pull it against the tooth and apply back and forth longitudinal movements until achieve the desired effect. Remove the strip.

Dimensions: 100mm x 4mm.

Warnings, precautions and restrictions: Professional use only. Single use only. Its reuse may cause cross contamination. When there is proven allergy to any of its component, the product must not be used. The strip in

contact with the gum can cause injuries to patient, we recommend the protection of the gum. The reuse may affect the product's performance. Any serious incident related to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

Storage: To be stored in a dry place, fresh and sheltered from sunlight.

Composition: Stainless steel.

Disposal: Dispose in accordance with applicable local requirements.

Ref.	Code	Content
0.3076	0009010101	10 strips of 100mm x 4mm

Resp. Técn: Gisele K. Gruber
CREA/SC 179557-7



Medical Device



EC REP Obelis S.A

Bd. Général Wahis 53
1030 Brussels, Belgium
Tel: +(32) 2.732.59.54
E-mail: mail@obelis.net

Manufacturer: TDV Dental Ltda.

CNPJ 81.591.786/0001-60
XV de Novembro, 9944
Pomerode/SC - Brasil
CEP 89107-000

☎ 55 47 3395-6115
0800 047 1020

✉ info@tdv.com.br
www.tdv.com.br
Reg. ANVISA 10291220068

IMP 0605000405 REV. 08 - 04/2023