EN SeptoDiamond Strips

Diamond steel strips for finishing of restorations and orthodontic stripping

SeptoDiamond Strips is a diamond steel strip indicated for finishing of filling materials and orthodontic stripping. Available in fine (yellow) and medium (red) grits.

Use: Insert the SeptoDiamond Strips in the interproximal area with the abrasive side turned to the surface to be worked (stripping or finishing). Conduct longitudinal movements, back and forth. The abrasive side is labelled according to the strip's grit: red (medium) and yellow (fine). To achieve the best results it is recommended to use both strips, starting with the medium grit and concluding with the fine grit.

Warnings, precautions and restrictions: Professional use only. Single use only. Its reuse can cause cross contamination. If there is proven allergy to any of its components, the product must not be used. The contact of the strip with the gum may cause minor injury to the patient. It is recommended to protect the papilla with a wedge or other protection method. Any serious incident related to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

Composition: Stainless steel, diamond and nickel.

Storage: To be stored in a cool and dry place, protected from sunlight.

Shapes: 2,5mm x 110mm

Disposal: Dispose in accordance with applicable local requirements.

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