SeptoPlus Mandrel / SeptoDiscs Mandrel

Accessory device for fixing different finishing and/or polishing systems in contra-angle. Available in two finishes: mandrel by pressure (SeptoPlus Mandrel) and pop-on mandrel.

Use: Install the finishing/polishing device on the mandrel. Make sure that the device is fully seated on the mandrel. Insert the mandrel shank into the chuck of the contra-angle handpiece. Perform a test run on the device to verify the product's effect on the workpiece. SeptoPlus Mandrel. Compatible with SeptoPolisher and SeptoWheels FX. SeptoDiscs Mandrel. Compatible with SeptoDiscs.

Warnings, precautions and restrictions: Professional use only. Only the use of the mandrel or discs with similar products is of the user's sole responsibility. To avoid an accidental swallowing of this product, it is necessary the application of the total isolation technique of the operating field. If there is proven allergy to any of the components, the product must not be used. Wear protective goggles to shield against particles (dentists and patients). Use of a rubber dam is highly recommended to avoid the potential ingestion of dental materials or particles. Perform a test run on the device to ensure that it will not handle any of the patient's denture. Contact with soft tissues (skin, gingiva, mucosa) may result in injury to the tissue. Any serious incident related to the device should be reported to the manufacturer or the competent authority of the Member State in which the user and/or patient is established.

Cleaning: Carry out the cleaning with the aid of water and enzymatic detergent 4 enzymes (Amylase, Protease, Lipase and Carbohydrase) and with pure pH range between 6,5 - 7,5. Prepare the dilution solution according to the manufacturer's instructions. The water should be potable. For best effectiveness, it is recommended that the temperature be between 40°C and 55°C. Submerge the product and leave in contact for 5 minutes with the solution. Remove parts and rinse under running water for at least 10 seconds. Dry the piece with absorbent paper or disposable wipers. **Sterilization:** The mandrel should be autoclaved (126° C, 16min), after each patient. **Reprocessing Limits:** Repetead processing has minimal effect on the mandrel due to its composition. This is a reusable device. SeptoPlus Mandrel. The manufacturer guarantees up to 50 reuses. SeptoDiscs Mandrel. The manufacturer guarantees 250 reuses. Carefully inspect the mandrel and discs after each use to ensure that all debris has been removed. Observe damage / breakage of the product that would prevent proper operation and do not use in this case.

Composition: SeptoPlus Mandrel: Metal nickel plated metal. SeptoDiscs Mandrel: Nickel plated metal.

Storage: Store below 30°C (86°F), in a dry place, protected from light. **Connection with contra-angle:** The SeptoPlus Mandrel / SeptoDiscs Mandrel is manufactured in compliance with specifications of ISO 1797/2017.

Reprocessing: SeptoPlus Mandrel - 1 unit. SeptoDiscs Mandrel - 1 unit. 011000005 - SeptoDiscs Mandrel - 1 unit

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